 DE - Sicherheitshinweise - Homebleaching	3
 EN - Safety Instructions - Home-Bleaching	3
 IT - Istruzioni di Sicurezza - sbiancamento domiciliare	4
 FR - Consignes de Sécurité - blanchiment dentaire à domicile.....	5
 ES - Instrucciones de Seguridad - blanqueamiento dental en casa.....	6
 SE - Säkerhetsanvisningar - tandblekning hemma.....	7
 PL - Instrukcje Bezpieczeństwa - domowe wybielanie	7
 NL - Veiligheidsinstructies - thuisbleekproducten	8

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EN - Safety Instructions - Home-Bleaching

General Warnings for Using Home-Bleaching Products

- ▶ **Use only after consulting a dentist:** Before using home-bleaching products, consult a dentist to ensure that your teeth and gums are healthy enough for bleaching.

- ▶ **Follow instructions carefully:** Adhere closely to the product's usage instructions, including the recommended application time and frequency.
- ▶ **Tooth sensitivity may occur:** Increased tooth sensitivity can occur during and after use. If sensitivity persists or becomes severe, discontinue use and consult a dentist.
- ▶ **Avoid contact with gums and soft tissues:** Bleaching gel should only be applied to the teeth and should not come into contact with the gums or other soft tissues in the mouth to prevent irritation.
- ▶ **Do not use with gum problems:** If you have inflamed or receding gums, avoid using bleaching products, as they could worsen these conditions.
- ▶ **Do not use on damaged teeth:** If you have cracks, cavities, or poorly filled teeth, bleaching gel can penetrate the tooth and cause pain or permanent damage.
- ▶ **Not suitable for children and pregnant women:** Home-bleaching products are generally not recommended for children under 18 or for pregnant or breastfeeding women.
- ▶ **Avoid acidic foods and drinks:** During and immediately after the bleaching treatment, avoid acidic foods and beverages, as teeth may be temporarily more sensitive.
- ▶ **Clean whitening trays thoroughly:** If a whitening tray is used to apply the bleaching gel, it should be cleaned thoroughly after each use to prevent bacterial buildup.
- ▶ **Results and duration may vary:** Whitening results vary from person to person and are not permanent. Regular touch-ups may be needed to maintain the effect.
- ▶ **Do not overuse:** Excessive bleaching can weaken enamel and make teeth more prone to cavities. Stick to the recommended intervals between treatments.

⚠ Always follow the instructions on the product or packaging.

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