	DE - Sicherheitshinweise - Interdentalbürsten.....	3
	EN - Safety Instructions -interdental brushes	3
	IT - Istruzioni di Sicurezza - scovolini interdentali.....	4
	FR - Consignes de Sécurité -brossettes interdentaires	5
	ES - Instrucciones de Seguridad -cepillos interdentaes	6
	SE - Säkerhetsanvisningar -mellanrumsborstar	7
	PL - Instrukcje Bezpieczeństwa -szczoteczki międzyzębowe	8
	NL - Veiligheidsinstructies - ragers	8

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EN - Safety Instructions -interdental brushes

⚠ General Warnings for Using Interdental Brushes ⚠

- ▶ **Choose the right brush size:** Select an interdental brush that fits snugly into your interdental spaces. A brush that is too small may not clean thoroughly, while one that is too large can injure the gums.

- ▶ **Gentle application:** Avoid applying excessive pressure to prevent gum injuries. Insert the brush gently into the interdental space.
- ▶ **Do not force into tight spaces:** If the brush does not fit easily into an interdental space, use a smaller size or consider an alternative cleaning method, such as dental floss.
- ▶ **Clean the brush regularly:** Rinse the interdental brush thoroughly with mouthwash or water after each use to remove bacteria and food particles.
- ▶ **Replace the brush regularly:** Depending on usage, replace interdental brushes every few days to weeks, or when the bristles become worn.
- ▶ **Caution with sensitive gums:** If your gums are sensitive or inflamed, proceed carefully or consult a dentist before using an interdental brush.
- ▶ **Monitor bleeding:** Slight gum bleeding may be normal at the beginning of use. If bleeding persists or is severe, consult a dentist.
- ▶ **Do not use with gum injuries:** Avoid using the brush if you have open wounds or injuries on your gums until they have healed.
- ▶ **Regular use:** Use interdental brushes daily to remove plaque and food debris from areas that a toothbrush cannot reach.
- ▶ **Interdental brushes for braces or implants:** If you have braces or implants, use specially designed brushes suitable for these purposes to prevent damage.
- ▶ **Use the correct technique:** Gently insert the brush into the interdental space without applying too much pressure, and move it back and forth several times to optimize cleaning.
- ▶ **Not a substitute for brushing:** Interdental brushes are a supplement to oral care and do not replace daily tooth brushing.

⚠ Always follow the instructions on the product or packaging.

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